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NOTICE TO BE READ BEFORE ALL USAGE

Before using the Magic Heatpack, read the precautions below

A. Advantages

This heat pack follows you everywhere: at home, at work or during your leisure. It warms instantly as soon as you wish: to relieve, to warm up or to relax. When put in the fridge, it offers you all the relief of cold.

B. Use hot

Flex the metal coin between 2 fingers in order to twist the coin. Clic! Malax for a few seconds so that the heat pack is soft and the heat is equally distributed. Apply on selected areas ; the heat will last between 30 minutes and 3 hours, depending on size and usage.

The heat pack gets solid after use.

To reactivate the warming properties of your heat pack, just place it in boiling water for 10 to 15 minutes, depending on size (do not turn off the fire under the pan, but regulate it on low heat). It is compulsory to protect your heat pad into a cloth during regeneration. After boiling, your heat pad will stay liquid and clear until next use.

Do not take the heat pack out of boiling water while there are still crystals in it: otherwise, it would become solid again after a few minutes.

After regeneration, either let it cold down, or use the warmth of the heat pack by using it as a traditional heat pack (caution the heat pad is very hot!) Once the Magic Heatpack is cold, it is ready to be clicked again.

C. Use cold


Protect the heat pack in a cloth or a freezer storage bag. Keep the heat pack in the fridge (5°/8° C) or place it in the freezer for around 10 minutes. Apply on selected areas.

For over more than 15 years, the Magic Heatpack has been the reference brand and never stops innovating for your comfort and well being. Often copied but never equalled, beware of imitations.

This heat pad has been produced with the utmost care, with materials selected for their durability and maximum security.

The Magic Heatpack contains tri-hydrated sodium acetate (CH₃-COONa-3H₂O).

Should you meet a problem within the framework of a normal use, we invite you to contact your usual retailer with your proof of purchase.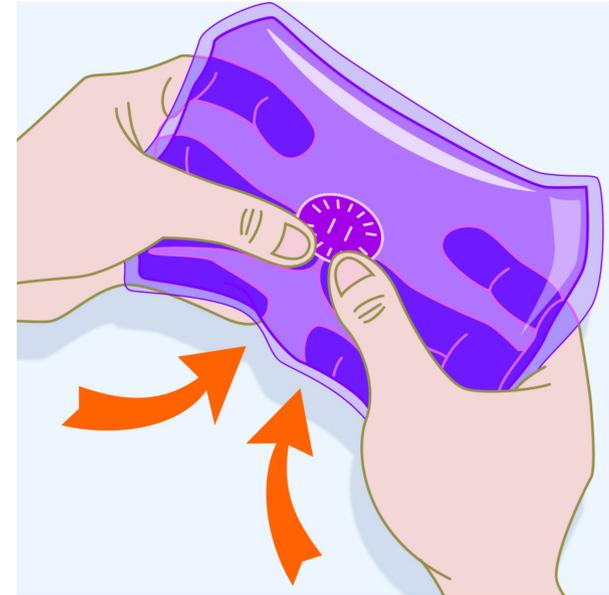

D. Usage precautions

- For external use only. The heat pad should be used only as described.
- Do not put in the microwave or in a traditional oven or grill.
- Make sure the heat pad is protected in a cloth during regeneration.
- Do not sit or lean on the heat pack or apply excessive pressure because it may cause the pouch to rupture.
- If you accidentally puncture your heat pack, rinse thoroughly skin or clothes with clear water.
- This product is not a toy: it should be used under the responsibility of an adult. Keep away from children or people with cognitive disabilities.
- Do not apply - cold or hot - directly on skin: use a soft cloth for protection.
- The heat pack should not be used if the outer pouch is damaged or shows signs of deterioration, or in case of leakage.
- Beware of burning when the heat pad is hot.
- Do not manipulate pins close to the heat pack, it might get punctured.
- Do not use in case of insensitivity to heat, circulation problems or varicose veins.
- Do not put in contact with areas of the skin that have been affected by a skin disease or an open wound.
- Do not ingest the content of the heat pack.
- If you notice that your heat pack gets solid again after regeneration, just put it again in hot water until there is no crystal left.